


Mammoview Markers

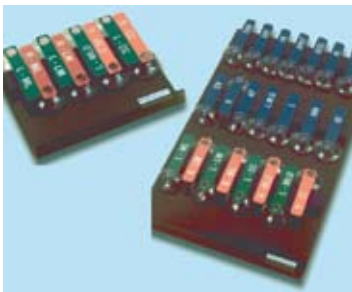

Model 021-022

Simple to use and compatible with any mammography X-ray system

CIRS radio-opaque markers are designed to provide a clear indication of position, which can be read directly from X-ray film. They are manufactured in accordance with abbreviations recommended by the ACR.

Each marker is manufactured to be clearly visible on X-ray film only in the mammographic energy range.

Each set of markers comes with an acrylic holding device designed to be mounted near the X-ray unit for easy access.

The CIRS Mammoview Markers are quick and easy to use. Simply mount the acrylic holder near the mammography unit in close proximity to the bucky. Firm pressure applied to the suction cup will hold the marker in place on any smooth surface. Mammoview Markers are usable on any mammography system.